

# How to start a NeurOptimal® Session

1

## Identify items.

Paste



zAmp & USB cable



Tablet/Computer



2

## Launch NeurOptimal®.

Launch NeurOptimal® by tapping on the NeurOptimal® icon in the Task Bar at the bottom of the screen



Vault




Media Player




3

## Connect USB cable to zAmp and Computer or Tablet.

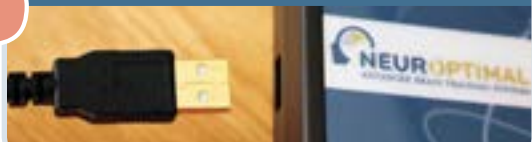

4


## Apply Sensors.

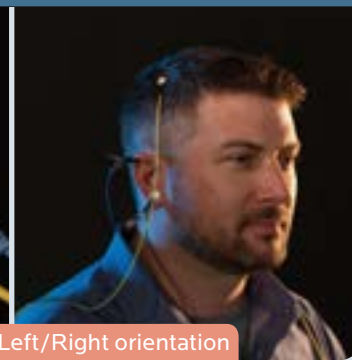

Note color coding and Left/Right orientation


5

## Double tap "Client" to add your name (if needed).

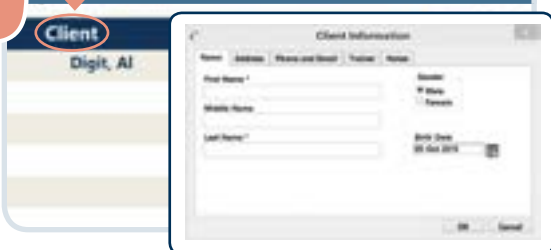

6

## Double tap your name to start a session.

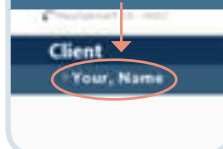

## IMPORTANT

Please always have your system connected to the internet.

Please contact your rental provider at \_\_\_\_\_ with any questions.